EECP® Clinical Results & Registry Data

Selected published results on the effects of Enhanced External Counterpulsation (EECP®) Therapy



What is EECP[®] Therapy?

EECP®/ECP Therapy is an FDA cleared, non-invasive medical therapy for the treatment of chronic stable angina that is refractory to optimal anti-anginal medical therapy without options for revascularization. In addition, based upon clinical studies published in peer reviewed medical journals, physicians also refer patients to EECP therapy for the relief of symptoms from other ischemic cardiovascular diseases such as left ventricular dysfunction or heart failure, diabetes, cerebrovascular disease, and peripheral vascular disease.

EECP® THERAPY DELIVERY

The EECP® Therapy system consists of three sets of inflatable pressure cuffs wrapped around the calves and the lower and upper thighs, including the buttocks. In synchronization with each cardiac cycle, obtained with an integrated 3-lead ECG, the cuffs are sequentially inflated from the calves to the buttocks during diastole to produce an arterial retrograde flow towards the aortic root to increase coronary blood flow. This inflation action also increases venous return, increasing blood volume to the ventricles and raises cardiac ouput. The cuffs are deflated simultaneously before the onset of systole, leaving behind an empty vascular space, reducing systemic vascular resistance, and significantly reducing the workload and oxygen demand of the heart.

PATIENT SELECTION

Currently, EECP® Therapy has been approved for national coverage by the Centers for Medicare and Medicaid Services (CMS) under the guidelines as a non-pharmacologic outpatient treatment for patients with chronic stable angina symptoms including chest pain, atypical pain, shortness of breath, fatigue, and cough. EECP® is also commonly used for the treatment of patients with heart failure.

More than 190 clinical papers have been published in peer-reviewed medical journals demonstrating EECP® to be safe and effective in the treatment of angina pectoris, chronic heart failure and other diseases.

CONTRAINDICATIONS

EECP® Therapy should not be used for the treatment of patients with:

- Arrhythmias that interfere with the machine triggering mechanism
- Bleeding diathesis
- Active thrombophlebitis
- Severe lower extremity vascular occlusive disease
- Presence of a documented aortic aneurysm requiring surgical repair
- Pregnancy

PRECAUTIONS

- Patients with blood pressure higher than 180/110 mmHg should be controlled prior to treatment with EECP[®].
- Patients with a heart rate more than 120 bpm should be controlled prior to treatment with EECP[®].
- Patients at high risk of complications from increased venous return should be carefully chosen and monitored during treatment with EECP®. Optimizing cardiac afterload reduction and cardiac output may help minimize increased cardiac filling pressures due to increased venous return.
- Patients with clinically significant valvular disease should be carefully chosen and monitored during treatment with EECP[®].

CLINICAL BENEFITS

Clinical evaluation of EECP® in patients with angina pectoris and congestive heart failure has been performed in multi-center, single center and registry-based clinical investigations. Results of these investigations have demonstrated clinical benefit and safety in:

- Time to ST-depression during stress test
- Peak oxygen consumption
- Exercise duration
- Angina episodes
- Nitroglycerin usage
- Quality of life
- Functional ability measures

MECHANISMS OF ACTION

In addition, the following mechanisms of action of EECP® have been demonstrated clearly in medical literature:

- Induces higher shear stress acting on the endothelium due to increased blood flow velocity
- Improves endothelial function by increasing nitric oxide and reducing endothelin-1
- Promotes angiogenesis and recruits collateral circulation to ischemic regions
- Reduces myocardial oxygen demand
- Inhibits inflammatory cytokines
- Reduces arterial stiffness
- Reduces the proliferation and migration of smooth muscle cells
- Inhibits the process of atherosclerosis

Evidence supporting these results are shown in the following selected papers.



MUST-EECP® (Multicenter Study of Enhanced External Counterpulsation): Effect of EECP® on Exercise Induced Myocardial Ischemia and Angina Episodes

Arora, et al., J Am Coll Cardiol 1999;33:1833-40

GOAL

Assess safety and efficacy of EECP[®]

ENDPOINTS


- Exercise duration
- Time to ≥1-mm ST-segment depression
- Angina episodes
- Nitroglycerin usage

STUDY DESIGN


- A multicenter, prospective, randomized, blinded, shamcontrolled trial conducted in 7 university hospitals with 139 angina outpatients randomized to the active EECP® group (n=72) using 300 mmHg pressure applied to the cuffs versus the inactive EECP® group (Control, n=67) with up to 75 mmHg pressure to the treatment cuffs
- Patients with documented angiographic stenosis
 >70% in at least one major coronary artery, history of myocardial infarction (MI), or positive nuclear exercise stress test for MI or ischemia
- Exercise treadmill test (ETT) using a standard or a modified Bruce protocol at baseline and within 1 week after completion of EECP[®] treatment
- 35 1-hour EECP[®] treatment sessions were delivered once or twice per day to both the active and the inactive group with appropriate cuff pressure.

RESULTS

- Significant increase in exercise time post-EECP[®] from baseline in the active EECP[®] group (426±20 to 470±20 s, p<0.001) versus inactive (432±22 to 464±22s, p<0.03). However, there was no statistically significant difference between the groups in change in exercise duration from baseline to post-EECP[®] (active 42±11 vs inactive 26±12 s, p>0.3).
- EECP[®] significantly improved time to ≥1-mm ST-segment depression in the active group (337±18 to 379±18 s, p<0.002) compared with the inactive group (326±21 to 330±20 s, p<0.74). There was a significant difference between the groups in the change in time to exercise-induced ischemia from baseline to post-EECP[®] in the active group (37±11 s) versus the inactive group (-4±12 s), p=0.01.

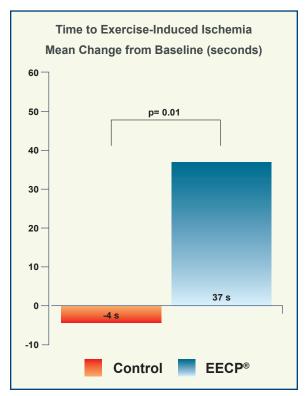

Angina episodes per day were 0.72±0.14 at baseline and 0.57±0.38 post-EECP® in the active group and 0.77±0.14 at baseline and 0.76±0.22 post treatment in the inactive group. The difference between the groups was significant with -0.033±0.27 in the active and 0.15±0.27 in the inactive group, p<0.035.

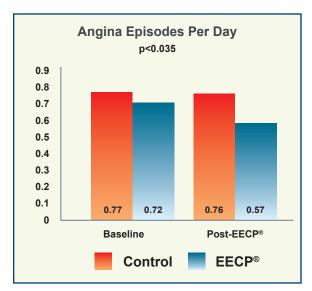

 Nitroglycerin usage was 0.39±0.11 and 0.56±0.17 at baseline, 0.12±0.04 and 0.43±0.21 post-EECP® in the active and the inactive group respectively. The difference between the groups was not significant, p>0.1.

ADVERSE EVENTS

 Leg discomfort including pain and abrasion was reported in 11.6±22.7% in the active group, and 4.9±18.7% in the inactive group.

CONCLUSION

 MUST-EECP® demonstrates that EECP® Therapy can reduce angina and extend the time to ischemia and increase exercise tolerance in patients with symptomatic CAD. PEECH (Prospective Evaluation of Enhanced External Counterpulsation in Congestive Heart Failure): EECP® Improves Exercise Tolerance in Patients with Chronic Heart Failure

Feldman, et al., J Am Coll Cardiol 2006;48:1198-205

GOAL

 Assess the benefits of EECP® in the treatment of patients with mild to moderate heart failure

ENDPOINTS

- Primary:
 - Percentage of subjects with ≥ 60 s increase in exercise duration at 6 months after EECP® treatment or
 - Percentage of subjects with ≥ 1.25 ml/min/kg increase in peak volume of oxygen uptake (pVO₂) at 6 months after completion of EECP® Therapy
- Secondary:
 - Exercise duration
 - Peak VO₂
 - NYHA functional classification
 - Quality of life using the Minnesota Living with Heart Failure (MLWHF) instrument

STUDY DESIGN


 A prospective, randomized, controlled trial conducted in 29 centers with NYHA functional class II and III heart failure patients (n=187*) having a left ventricular ejection fraction (LVEF)≤ 35%, randomized in a 1:1 ratio to EECP® (n=93) versus control with protocol-defined pharmacologic therapy (PT, n=94)

* Intent to treat


- Exercise treadmill test (ETT) and peak oxygen uptake using a standard modified Naughton protocol at baseline, 1 week after EECP® treatment completion and at 3-month and 6-month follow-up
- 35 1-hour EECP® treatment sessions were delivered once or twice per day to the EECP® group.

RESULTS


- Exercise duration increased by ≥ 60 s in 35.4% in the EECP[®] group compared with 25.3% in the PT group at the 6-month follow-up visit (p=0.016).
- Peak VO₂ increase by ≥ 1.25 ml/kg/min did not differ between the two groups (22.8% vs 24.1%).

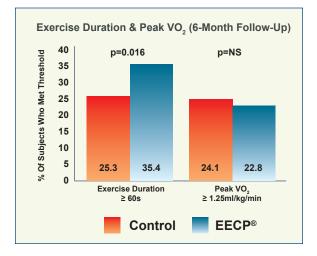


 EECP® was associated with a significant increase in exercise duration versus PT at 1-week follow-up (26.4±12.2 vs -5.5±11.7 s, p<0.01), at 3-month followup (34.5±13.9 vs -7.0±12.7 s, p=0.014) and at 6-month follow-up (24.7±15.2 vs -9.9±13.2 s, p=0.013) after treatment.

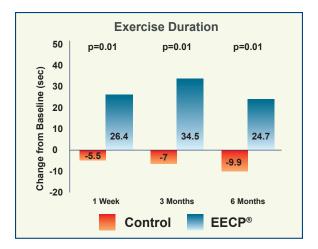

 EECP® significantly improved ≥ 1 NYHA functional class with 33.3% of patients vs 11.4% in the PT group 1 week after completion of EECP® treatment, and 31.6% vs 12.2% at 3 month, 31.3% vs 14.3% at 6-month follow-up.

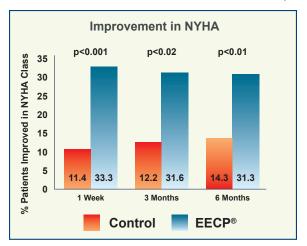

 Quality of Life (QoL), assessed by a Minnesota Living with Heart Failure instrument, also improved significantly in the EECP® group when compared with the PT group at the 1-week and 3-month followup. However, there was no difference in QoL at the 6-month follow-up.

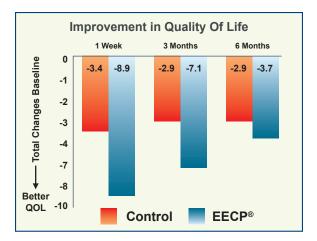

ADVERSE EVENTS

- Adverse events occurred in 11.8% of the EECP® group versus 3.2% in the PT group.
- However, the number of major adverse cardiovascular events (MACE) that occurred during the trial were small and were the same in both the EECP® and the PT groups.

CONCLUSION

 EECP® Therapy improved exercise tolerance and HF symptoms in patients with NYHA class II and III HF, but did not improve peak VO₂ 6 months after completion of EECP® treatment. EECP® Improves Exercise Duration and Peak Oxygen Consumption in Older Patients with Heart Failure: A Subgroup Analysis of the PEECH Trial

Abbottsmith, et al., CHF 2006;12:307-311

GOAL

 Assess whether the effects of EECP® Therapy in the overall PEECH population could be observed in patients 65 years and older

ENDPOINTS


- Pecentage of subjects with ≥ 60 s increase in exercise duration at 6 months after completion of EECP® Therapy or
- Percentage of subjects with ≥ 1.25 ml/min/kg increase in peak volume of oxygen uptake (pVO₂) at 6 months after completion of EECP® therapy


STUDY DESIGN

 A prespecified subgroup of elderly patients ≥ 65 years enrolled in the PEECH trial were randomized to the EECP® group (n=41) versus the control with protocoldefined pharmacologic therapy (PT, n=44) group

RESULTS

In contrast to the overall population in the PEECH trial, the major findings of this subgroup analysis demonstrated a significantly higher responder rate in pVO₂ in the EECP® group at the 6-month follow-up when compared with the control PT group (29.7% vs 11.4%, p=0.017).

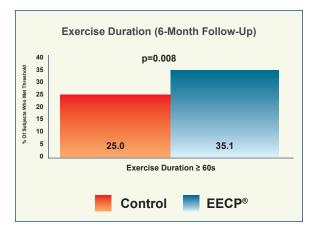

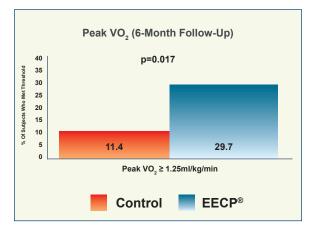

ADVERSE EVENTS

 MACE occurred in fewer subjects in the EECP® group relative to the PT group (29.3% vs 36.4%).

CONCLUSION

 The results of this subgroup analysis indicate that adjunctive use of EECP® Therapy provides functional benefit in older patients with chronic, stable systolic HF in contrast to the overall population in the PEECH study. The EECP® group demonstrated a significantly higher rate of pVO₂ improvement than the control group.

Coronary Collateral Growth by External Counterpulsation: A Randomized Controlled Trial

Gloekler, et. al, Heart 2010;96:202-207

GOAL

 To test the hypothesis that EECP® treatment augments coronary collateral development

ENDPOINTS


- Primary:
 - Coronary collateral flow index (CFI)
- Secondary:
 - Coronary collateral conductance (CCC)
 - Brachial artery flow-mediated dilatation (FMD)

STUDY DESIGN

- 20 sessions of EECP® Therapy over 4 weeks, each session lasting 90 minutes, were performed in 20 chronic stable coronary artery disease patients randomly assigned to the EECP® group (n=10) with 300 mmHg cuff inflation pressure versus the sham control group (n=10) with 80 mmHg inflation pressure.
- CFI was determined by invasive coronary catheterization pressure measurements and calculated by dividing the mean distal coronary pressure during balloon occlusion by the mean aortic pressure after subtracting out the central venous pressure (CVP) from both.
- CCC was determined by the ratio of myocardial blood flow to the difference of aortic pressure to distal coronary pressure during balloon occlusion.
- FMD was determined using 2-dimensional vascular ultrasound imaging.

RESULTS

 CFI changed from 0.125±0.073 at baseline to 0.174±0.104 at follow-up in the EECP® group (p=0.006), and from 0.129±0.122 to 0.111±0.125 at follow-up in the sham group (p=0.14). The absolute change in CFI from baseline to follow-up amounted to 0.069±0.128 in the EECP® group and -0.017±0.049 in the sham group (p=0.0009).

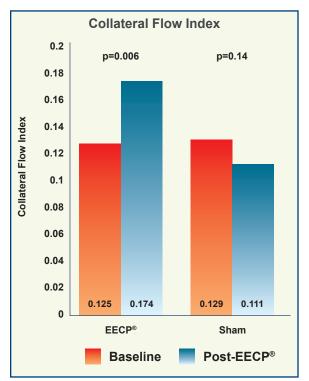

 Resting CCC obtained during vessel occlusion changed from 0.365±0.268 at baseline to 0.568±0.585 ml/ min/100 mmHg at follow-up in the EECP® group (p=0.072), and from 0.229±0.212 at baseline to 0.305±0.422 ml/min/100 mmHg at follow-up in the sham group (p=0.45). FMD changed from 4.3±1.5 % at baseline to 6.9±3.5 % at follow-up in the EECP® group (p=0.018), and from 6.0±3.0 % at baseline to 7.6±3.5 % at follow-up in the sham group (p=0.10). The absolute change in FMD from baseline to follow-up amounted to 1.75±2.8 % in the EECP® group and 0.50±1.0 % in the sham group (p=0.07).

ADVERSE EVENTS

No MACE was reported.

CONCLUSION

 EECP® Therapy appears to be effective in promoting collateral growth. The extent of collateral function improvement is related to the amount of improvement in systemic endothelial function. EECP® Improves Peripheral Artery Flow-Mediated Dilation in Patients with Chronic Angina: A Randomized Sham-Controlled Study

Braith, et al., Circulation 2010;122:1612-1620

GOAL

 To investigate the effects of EECP® on peripheral artery flow-mediated dilation, endothelial functions and clinical outcomes

ENDPOINTS

- Endothelial function: brachial artery flow-mediated dilatation (FMD), nitric oxide nitrate/nitrite (NOx), 6-keto-prostaglandin F₁₀ (PGF₁₀) and endothelin-1 (ET-1)
- Inflammatory markers: tumor necrosis factor-a (TNF-a), high sensitivity C-reactive protein (hs-CRP), monocyte chemoattractant protein-1 (MCP-1), and soluble vascular cell adhesion molecule (sVCAM)
- Lipid peroxidation marker: 8-isoprostane-F_{2a} (PGF_{2a}), and asymmetrical dimethylarginine (ADMA)
- Clinical outcomes: blood pressure, exercise duration, peak oxygen consumption (peak VO₂), Canadian Cardiovascular Society (CCS) functional class, angina frequency and nitrate usage

STUDY DESIGN

 42 consecutive patients with chronic stable angina were randomized in a 2:1 manner into either EECP® treatment (n=28) with 300 mmHg cuff inflation pressure or sham group (n=14) with 70 mmHg inflation pressure.

- EECP® treatment was given in 35 hourly sessions over 7 weeks.
- FMD was determined using 2-dimensional vascular ultrasound imaging.
- Symptom-limited exercise tests using a modified Naughton protocol were performed by all subjects before and after 35 sessions of EECP[®].

RESULTS

- There were significant improvements in endothelial function, reductions in inflammatory markers and lipid peroxidation in the EECP® group with no significant changes in the sham group.
- Evidence of improved arterial FMD and endothelialderived vasoactive agents after EECP® treatment was paralleled by decreased angina symptoms and reduced CCS angina classification from a mean of 3.1 to 1.2 (p<0.001).

	EECP®	Sham	р
FMD	51% 🕇	2% 🕇	<0.05
NOx	36% 🕇	2% 🕇	<0.05
6-keto-PGF _{1α}	71% 🕇	1% 🕈	<0.05
ET-1	25% 🕇	5% †	<0.05
TNF-a	16% 🕇	12% 🕇	<0.05
hs-CRP	32% 🕇	5% †	<0.05
MCP-1	13% 🕇	0.2% 🕇	<0.05

	EECP®	Sham	р
sVCAM	6% 🕇	1% 🕈	<0.05
8-iso-PGF _{2α}	21% 🕇	1.3% 🕇	<0.05
ADMA	28% 🕇	0.2% 🕇	<0.05
Exercise Duration	24% 🕇	3% 🕇	<0.001
Peak VO ₂	12.3% 🕇	0.6% 🕇	=0.002
CCS Class	62% 🕇	0% 🕈	<0.001
Angina Frequency	72% 🕇	6% 🕇	<0.001
Nitrate Usage	81% 🕇	10% 🕇	<0.01

ADVERSE EVENTS

No MACE was reported.

CONCLUSION

 EECP® Therapy demonstrates in CAD patients to have beneficial effects: improvement in brachial & femoral artery FMD, increases in bioavailability of nitric oxide and PGF₁₀, decreases in plasma inflammatory and oxidative stress biomarkers. These beneficial changes were in parallel with improvements of clinical outcomes such as a decrease in angina symptoms and an increase in exercise tolerance. Effects of Enhanced External Counterpulsation on Arterial Stiffness and Myocardial Oxygen Demand in Patients with Chronic Angina Pectoris: A Randomized Sham-Controlled study

Casey, et al., Am J Cardio. 2011;107(10):1466-1472

GOAL

 To test the hypothesis that decreases in arterial stiffness and aortic wave reflection are a therapeutic target related to benefits in clinical outcomes

ENDPOINTS

- Central and peripheral pulse-wave velocity (PWV)
- Aortic wave reflection augmentation index (Alx)
- Wasted left ventricular pressure energy
- Aortic systolic tension-time index (AsTTI (mmHg*s*min-1))
- Clinical outcomes: exercise duration, peak time to angina, peak oxygen consumption (peak VO₂), and Canadian Cardiovascular Society (CCS) functional class, angina frequency and nitrate usage


STUDY DESIGN

- 42 consecutive patients with chronic stable angina were randomized in a 2:1 manner into either the EECP® treatment group (n=28) with 300 mmHg cuff inflation pressure or the sham group (n=14) with 70 mmHg inflation pressure.
- All data was taken at baseline within 1 week before EECP® treatment, then at the 17th hour and at the completion of 35 hours.


- EECP® treatment was given in 35 hourly sessions over 7 weeks.
- Assessment of arterial wave reflection was performed with the SphygmoCor system recording peripheral pressure waveforms using applanation tonometry with a high-fidelity micromanometer. Results of the peripheral measurements were used to generate the corresponding aortic waveform using a generalized transfer function.
- A SphygmoCor Pulse Wave Velocity Vx system was used to measure PWV at 3 sets of sites sequentially: carotidradial, carotid-femoral and femoral-dorsalis pedis.
 Central PWV was evaluated using the carotid-femoral data and peripheral PWV using the femoral-dorsalis pedis and carotid-radial data.
- Symptom-limited exercise tests using a modified Naughton protocol were performed by all subjects before and after 35 hours of EECP® treatment.

RESULTS


 Carotid-femoral PWV was decreased at the 17th hour (10.6±0.4 m/s) and post-EECP® (10.2±0.4 m/s) compared to baseline (11.5±0.5 m/s), both p<0.01, while there were no changes in the sham group.

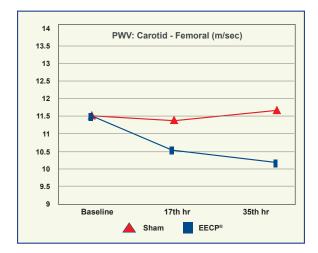


 PWV in the Carotid-radial (7.8±0.3 vs 8.3±0.3 m/s, p<0.01) and femoral-dorsalis pedis (8.7±0.4 vs 9.9±0.4 m/s p<0.05) were significantly decreased post-EECP® compared to baseline, while there were no changes in the sham group.

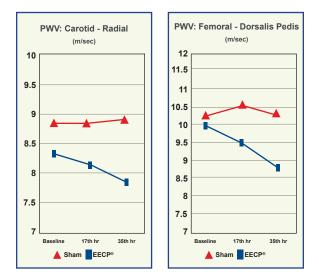


 EECP[®] produced a progressive decrease in Alx of 25.8±2.7% at the17th hour and 23.3±2.7% at the 35th hour compared to 29.1±2.3% at baseline (p≤0.01) while there were no changes in the sham group.

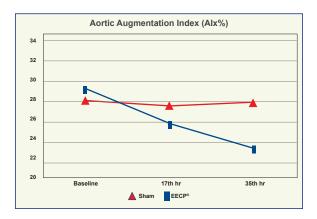


 There was a substantial decrease in wasted left ventricular pressure energy after 35 hours of EECP® treatment compared to baseline (1,598±233 vs 2,167±372 dyne/cm2/s, p<0.01).

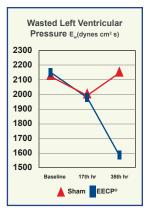


 EECP® treatment led to progressive increases in pVO₂ from 17.0±1.3 at baseline to 18.4±1.4 after 17 hours and 19.4±1.5 ml/ kg/ min at 35 hours; while there were no changes in the sham group, 16.5±1.3, 17.0±1.4 and 16.6±1.4 respectively.

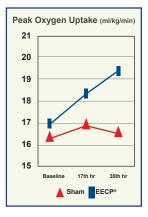


 Total exercise duration also increased significantly in the EECP® group from 586±41 at baseline to 678±52 after 17 hours and 774±65 sec after 35 hours while there were no changes in the sham group.

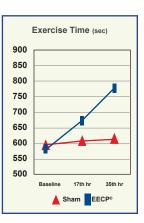


- Peak time to angina increased significantly in the EECP® group from 406±39 at baseline to 537±48 after the 17th hour, and 645±64 sec after the 35th hour, while there was no change in the sham group (449±54, 466±52 and 471±53 sec respectively).
- There were statistically significant decreases in CCS class from 3.2±0.1 at baseline to 2.4±0.1 after 17 hours and 1.2±0.1 after 35 hours, while there were no changes in the sham group (2.9±0.1, 2.9±0.1 and 2.9±0.1).

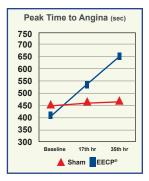

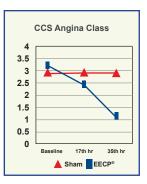

 There were also statistically significant decreases in the average angina episodes per day and nitroglycerin usage per day in the EECP® group while there were no significant changes in the sham group.

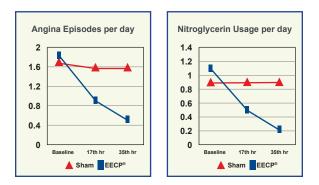

ADVERSE EVENTS

No MACE was reported.

CONCLUSION

 EECP® Therapy decreases central and peripheral arterial stiffness, which may explain improvements in myocardial oxygen demand in patients with chronic angina pectoris after treatment. These changes were parallel to improvements in clinical outcomes including exercise tolerance, peak oxygen consumption and angina functional class.

Art.Net.-2 Trial: Improvement of Fractional Flow Reserve and Collateral Flow by Treatment with EECP®

E. E. Buschmann, et al., Eur J Clin Invest 2009;39 (10):866-875

GOAL

 To investigate the effect of EECP® on coronary collateral artery growth

ENDPOINTS

- Primary: Invasive pressure derived collateral flow index (CFIp) and fractional flow reserve (FFR)
- Secondary: Symptom-limited bicycle ergometric test
- Canadian Cardiovascular Society (CCS) and New York Heart Association (NYHA) classification


STUDY DESIGN

- 23 patients with angiographic narrowing of >70% in at least one coronary artery assessed by FFR<0.8 were randomized in a 2:1 manner to 35 hours, 1 hour daily, 5 hours per week for 7 weeks active EECP® treatment (n=16) or to the control group (n=7) with 7 weeks, 5 times per week walk-in appointment for non-therapy related diagnostics, nutrition-counseling.
- Cardiac catheterization with hemodynamic measurements of aortic pressure (Pa), central venous pressure (Pv), mean distal coronary pressure (Pd) and coronary wedge pressure (Pw) were measured with a balloon occlusion proximal to the coronary stenosis.

- A symptom-limited bicycle ergometric test was performed at baseline and at the 8th week starting with 25 or 50 Watts and continued with an increase of 25 Watts every 2 minutes.
- CCS and NYHA were evaluated before the invasive procedure at baseline and at the 8th week.

RESULTS

- CFIp defined to be the ratio of (Pw-Pv) to (Pa-Pv) increased from 0.08±0.01 to 0.15±0.02 (p<0.001) in the EECP[®] group while there was no significant change in the control group (0.15±0.03 to 0.14±0.02 (p=0.7).
- The FFR changed from 0.68±0.03 at baseline to 0.79±0.03 at follow-up in the EECP® group (p=0.001), with no change in the control group from 0.68±0.06 to 0.7±0.05 (p=0.4) at follow-up.

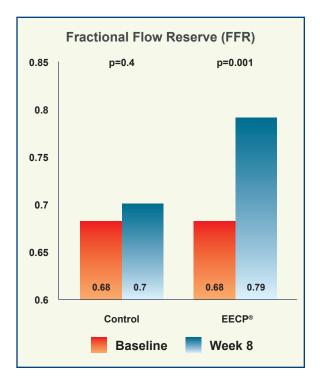

- A significant reduction in CCS classification was achieved after EECP® treatment (p=0.008) whereas in the control group no change was observed (p=0.25).
- The severity of dyspnea as measured by NYHA classification was reduced after EECP® (p<0.001) but not within the control group (p=0.28).
- At the end of therapy, 81% of the EECP® group were free of angina pectoris compared to 56% at baseline, and in the control group, only 14% improved after 7 weeks.

ADVERSE EVENTS

• No MACE occurred during the study.

CONCLUSION

 This study provides direct functional evidence for the stimulation of coronary angiogenesis via EECP[®] in patients with stable CAD.

Cyclic GMP Release by Acute Enhanced External Counterpulsation: A Randomized Sham-Controlled Study

Levenson, et al., Am J Hypertension 2006;19:867-872

GOAL

- To investigate the effect of a single 1-hour session of EECP® treatment on plasma and platelet cGMP
- Determine whether the activation of nitric oxide dependent pathways modulate changes in platelet cGMP

ENDPOINTS


- Plasma and platelet cGMP
- Inhibition of nitric oxide synthesis by Ng-monomethyl-L-arginine (L-NMMA) on platelet cGMP
- Effect on platelet cGMP by neutralizing superoxide anion with superoxide dismutase (SOD) in the presence of catalase
- Inhibition of phosphodiesterase 5 by Zaprinast on platelet cGMP

STUDY DESIGN


 55 subjects, 25 asymptomatic with high cardiovascular risk (HCVR) and 30 with angiographically proven coronary artery disease (CAD), were randomized to sham EECP® treatment (n=28) with less than 75 mmHg applied to cuff inflation with the remaining 27 subjects enrolled in the active EECP® group with 300 mmHg inflation pressure. Blood samples were collected before and after 1 hour of active or sham EECP® treatment and plasma and platelet cGMP levels were measured by radioimmunoassay.

RESULTS

 Absolute changes in plasma cGMP doubled in CAD subjects undergoing active EECP® treatment compared with sham control (p<0.01) by 235% in active EECP® CAD patients but not in the group with HCVR.

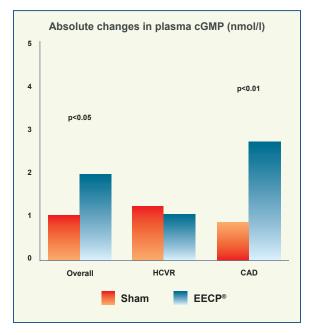

- Application of 1 hour of active EECP® treatment significantly increased platelet cGMP content by 19% (p<0.01) in the overall population (CAD+HCVR) but not in the sham group.
- EECP® significantly increased platelet cGMP compared to their respective baseline values (by 22% for L-NMMA, by 21% for SOD with catalase, and by 63% after inhibition of phosphodiestarase 5, all p<0.01).

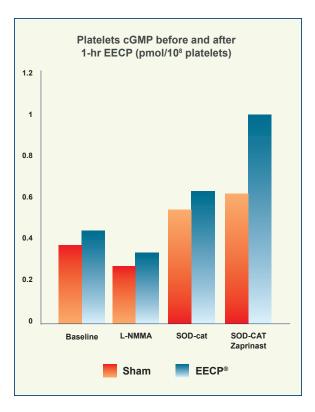

ADVERSE EVENTS

No MACE was reported.

CONCLUSION

- The counterpulsation induced cGMP increase was twice as large in subjects receiving active EECP® treatment versus those serving as a sham group.
- EECP® increases platelet cGMP content, which suggests nitric oxide synthase activation. EECP® increases cGMP production which may be the mechanism by which EECP® exerts its clinical benefit.
- Analysis of the modulation of platelet cGMP content suggests EECP® activates the nitric oxide-dependent pathways.

Effects of EECP® on Carotid Circulation in Patients with Coronary Artery Disease

Levenson, et al., Cardiology 2007;108:104-110

GOAL

 A randomized, sham-controlled study to investigate whether arterial stiffness and resistance of carotid circulation are altered by EECP® Therapy

ENDPOINTS

- The ß stiffness index
- Carotid vascular resistance


STUDY DESIGN

- 30 patients with stable coronary artery disease (CAD) demonstrated by angiographic proven stenosis greater than 50% in at least one major coronary artery or a documented MI treated according to medical therapy guidelines were randomized in a 1:1 manner into either EECP® treatment (n=15) with 300 mmHg cuff inflation pressure or sham group (n=15) with 75 mmHg inflation pressure.
- All data was taken at baseline within 1 week prior to EECP® treatment, then at the 17th hour and at the completion of 35 hours, 1 hour daily for 7 weeks.
- Echo-Doppler studies were performed with a highresolution ultrasound imager on the right and left common carotid arteries with a 7.5 to 12 MHz probe.
- The ß stiffness index calculated by ln(Ps/Pd)/ DD, where Ps and Pd are the systolic and diastolic blood pressure and DD is the ratio between carotid pulse and diastolic diameter.
- Carotid vascular resistance was calculated as the ratio between mean arterial blood pressure and mean blood


flow (mmHg/ml/sec).

RESULTS

 The ß stiffness index was significantly reduced at 1 hour (-3.7, p<0.01), 17 hours (-3.6, p<0.001) and 35 hours (-3.3, p<0.01) in the active EECP® group, and this decrease was statistically different when compared with the sham group at 1 hour and 35 hours (p<0.05 respectively).

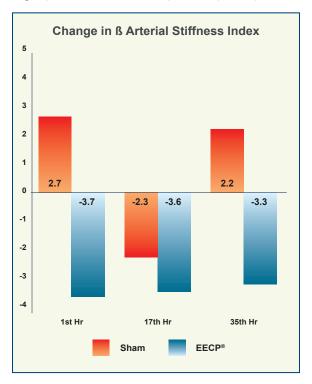

 Carotid vascular resistance was concomitantly reduced in the active EECP® group at 17 hours (-2.1 mmHg/ml/ sec, p<0.05), and 35 hours (-3.9 mmHg/ml/sec, p<0.01), while there was an increase in the sham group showing a significant difference between groups at 17 hours (p<0.01) and 35 hours (p<0.001) respectively.

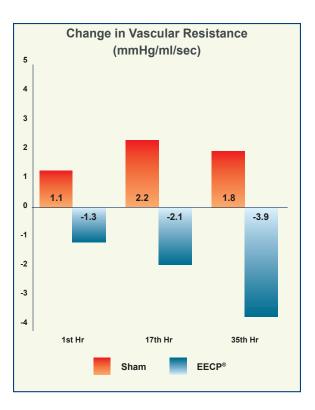

ADVERSE EVENTS

 One patient with worsening arrhythmia withdrew from the sham EECP[®] group after 17 hours. No other major complications occurred.

CONCLUSION

• EECP® exerts clear vascular relaxation effects on both large and small arteries of carotid circulation.

Left Ventricular Systolic Unloading and Augmentation of Intracoronary Pressure and Doppler Flow during EECP®

Michaels, et al., Circulation 2002;106:1237-1242

GOAL

 Examine intracoronary and left ventricular hemodynamics in the cardiac catheterization laboratory during EECP®

ENDPOINTS

- Diastolic and systolic pressure in the central aorta during EECP®
- Diastolic and systolic pressure in the coronary artery during EECP®
- Intracoronary average peak Doppler velocity during EECP[®]
- Peak diastolic blood flow velocity in the coronary artery during EECP®


STUDY DESIGN

 10 outpatients referred for diagnostic cardiac catheterization with suspected coronary artery disease (n=5), severe mitral regurgitation (n=3), and prior orthotopic heart transplantation (n=2) were studied with measurement of the central aortic pressure, intracoronary pressure and intracoronary Doppler flow velocity with a sensor-tipped transducer and guidewire.


RESULTS

 Significant decrease in peak aortic systolic pressure by 11% from 114±19 at baseline to 101±28 mmHg during EECP® (p=0.02)

- Significant increase in aortic diastolic pressure by 92% from 71±10 at baseline to 136±22 mm Hg during EECP® (p<0.0001)
- Significant increase in intracoronary peak diastolic pressure by 93% from 71±10 at baseline to137±21 mm Hg during EECP® (p<0.0001)

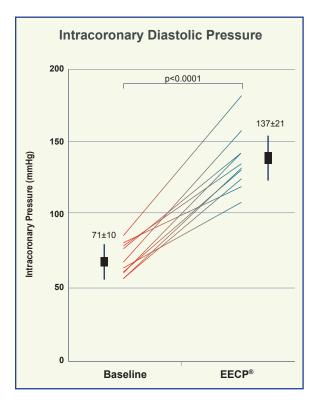

- Significant decrease in peak systolic intracoronary pressure by 15% from a baseline of 116±20 to 99±26 mm Hg during EECP® (p=0.002)
- Significant increase in intracoronary average peak Doppler velocity by 109% from 11±5 at baseline to 23±5 cm/s during EECP® (p=0.001)
- Significant increase in intracoronary peak diastolic blood flow velocity by 150% from 18±7 at baseline to 45±14 cm/s during EECP® (p<0.0004)

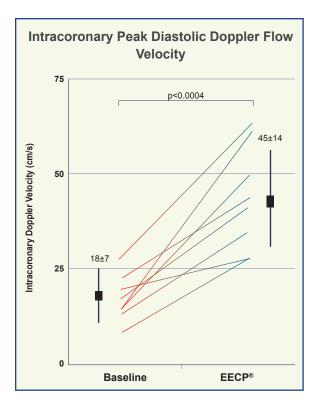

ADVERSE EVENTS

• No MACE was reported during the study.

CONCLUSION

- EECP® significantly decreases aortic systolic pressure to reduce the workload of the heart, resulting in decreased myocardial oxygen demand.
- EECP® significantly increases aortic and intracoronary diastolic pressure, thereby increasing intracoronary blood flow, augmenting energy supply to the myocardium.

International EECP® Patient Registry I and II (IEPR I and II)

GOAL

 IEPR I and II were organized to document the patterns of use, patient characteristics, safety and efficacy of EECP® during the treatment period and long term follow-up for a period of 3 years post treatment.

STUDY DESIGN

- IEPR-I: 5,056 patients from Jan 1998 to July 2001 in 119
 U.S. and 21 International sites with a 3 year follow-up
- IEPR-II: 2,917 consecutive patients from Jan 2002 to Oct 2004 in 95 U.S. sites with a 2 year follow-up
- IEPR data was collected by the Epidemiology Data Center of the University of Pittsburgh.
- Entry criteria: consecutive patients gave consent and underwent at least 1 hour of EECP® treatment.

DATA COLLECTION

- IEPR-1
 - Patients' demographics
 - Medical history
 - CAD status, quality of life
 - CCS Classification, medication
 - Angina frequency
 - Adverse clinical events before EECP®, post EECP®, and during follow-up period
- IEPR-2
 - NYHA class
 - Number of hospitalizations for heart failure patients
 - Duke Activity Status Index

 Responses to the Kansas Cardiomyopathy Questionnaire


RESULTS

(Selected results from more than 66 published papers)

Primer: Practical Approach to the Selection of Patients for and Application of EECP®

Michaels, et al., Nat Clin Pract Cardiovascular Med 2006;3(11):623-632


 At baseline, 86% of the 7,973 CAD patients were in Canadian Cardiovascular Society (CCS) angina class III and IV. At the 6-month follow-up after EECP®, 25% of the 4,565 patients were in CCS class III and IV. 76% of patients improved their CCS angina status by at least 1 class.

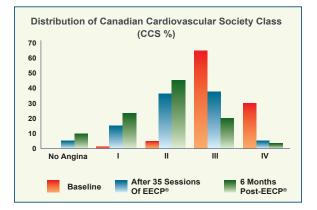

Enhanced External Counterpulsation in the Treatment of Chronic Refractory Angina: A Long-term Follow-up Outcome from the International Enhanced External Counterpulsation Patient Registry

Loh, et al., Clin Cardiol. 2008;31,4:159-164

 At 3 years follow-up, there was a reduction in CCS Class of at least 1 class in 78% of the patients, and by at least 2 classes in 38%. This was sustained in 74% of the patients during follow-up.

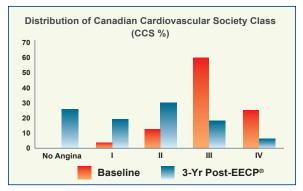

 The frequency of angina reduced by 4-6 episodes per week immediately post-EECP® and this was sustained at the 3 year follow-up.


ADVERSE EVENTS

- During EECP® treatment, 1.5% experienced musculoskeletal discomfort, 2.5% skin breakdown, 2.2% heart failure exacerbation, 4.5% unstable angina, 0.8% MI, 0.8% PCI, 0.6% CABG and 0.5% death.
- Cumulative 3-year mortality was 17%, MI was 11.8%, and MACE was 40.8%.

Impact of EECP® on Emergency Department Visits and Hospitalizations in Refractory Angina Patients with Left Ventricular Dysfunction

Soran, et al., CHF 2007;13:36-40

- There were 2,917 patients enrolled in IEPR-II. 450 had refractory angina with LV dysfunction (LVEF≤40%) and complete data on emergency department (ED) visits and hospitalizations 6 months before EECP® treatment. 93% were in CCS class III and IV, 50% had history of heart failure (HF), mean LVEF was 30±8%.
- After completion of EECP® treatment, angina class decreased by at least 1 CCS class in 72% of the patients, 19% reported no angina and 2% had an increase in angina class. Mean angina frequency decreased by 7±14 episodes per week from 11.4±16.9 to 3.8±10.9 (p<0.001).

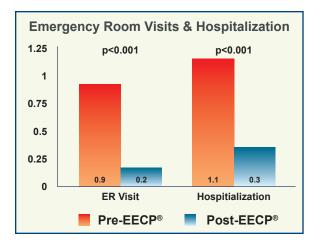

 The mean number of ER visits per patient decreased from 0.9±2.0 during the 6-month period pre-EECP® to 0.2±0.7 during the 6-month period post-EECP® (p<0.001). The mean number of hospitalizations reduced from 1.1±1.7 during the 6-month period pre-EECP® to 0.3±0.7 during the 6-month period post-EECP® (p<0.001).

ADVERSE EVENTS

- MACE occurring during EECP® treatment was 0.7% MI, 0.2% CABG, 1.2% PCI, and no death for a total MACE rate of 1.8%. Heart failure exacerbation was reported in 2.6% of patients.
- 6 months after completion of EECP® treatment cumulative mortality was 5.3%, MI was 1.9%, and MACE was 10.6%.

CONCLUSION

- EECP® used in everyday clinical practice without the constraint of trial inclusion and exclusion criteria demonstrated improvement in angina class in 74% of the treated patients and these benefits were sustained for up to 3 years.
- Similarly, EECP® reduced the number of angina episodes.
- EECP® Therapy was cost effective in reducing the number of emergency department visits and hospitalizations.

Learn More About EECP® Therapy

WWW.EECP.COM

EECP® Therapy information for patients and medical professionals

WWW.EECPFORUM.COM

Discuss EECP® Therapy with patients, therapists and physicians on the message board

FREE IPHONE APP FOR EECP® THERAPY



Conveniently provides patients, physicians and EECP® therapists with the information necessary for optimal EECP® therapeutic care Scan the QR Code with your iPhone to download your FREE application now!

WWW.VASOMEDICAL.COM

Corporate Information about Vasomedical, Inc., a world leader in the non-invasive treatment of cardiovascular disease

SHOP.VASOMEDICAL.COM

Easy online ordering of EECP® accessories and supplies, patient management products and promotional items



180 Linden Avenue • Westbury, NY 11590 Phone: 800-455-EECP (3327) Fax: 516-997-2299 customerservice@vasomedical.com



© 2014. Vasomedical, Inc. EECP® is a registered trademark of Vasomedical, Inc. All Rights Reserved. V15-0005 R.6